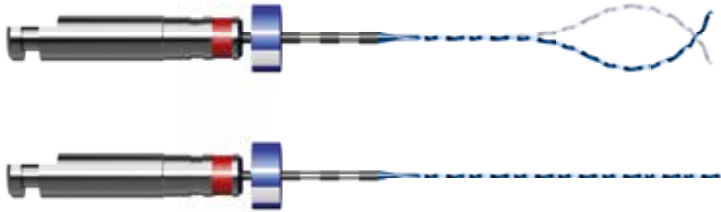
# XP-endo Finisher

Protocolo  
Protocolo  
Protocol

## ► Reglas de oro

800 rpm  
1 Ncm

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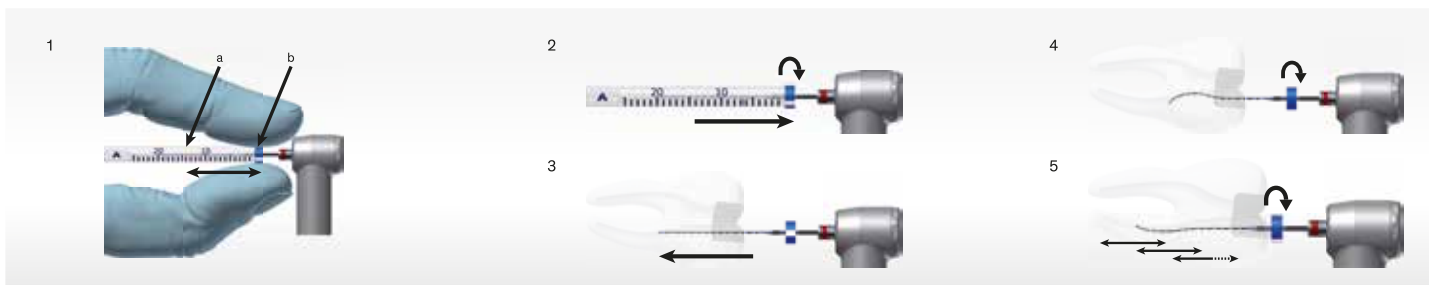
## ► Golden rules

800 rpm  
1 Ncm

- XP-endo Finisher must only be used only after preparing the canal to at least #25;
- In multiradicated teeth, begin with the largest canal;
- The access cavity must be filled with irrigant only after inserting the XP-endo Finisher into the canal.

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## ► Protocol

800 rpm (800-1000)  
1 Ncm

- Set the working length by positioning the rubber stop (b) using the plastic tube (a) (Figure 1).
- Cool the XP-endo Finisher with a cooling spray via the tube.
- Switch the XP-endo Finisher to rotate mode and press against the side walls as you remove it from the tube to ensure that the file is straight (Figure 2). Then deactivate rotate mode.
- Insert the straightened XP-endo Finisher into the first tooth canal (Figure 3) and, once the tip has been inserted, activate rotate mode (Figure 4). Add some irrigant to the access cavity.
- Use the XP-endo Finisher for approximately one (1) minute, applying slow, gentle longitudinal movements of 7-8 mm, so as to cover the entire length of the canal (Figure 5). Press the instrument against the side walls of the canals during the procedure. Make sure to stay inside the canal.
- After one minute, remove the XP-endo Finisher from the canal, still in rotation mode.