
SAB 1000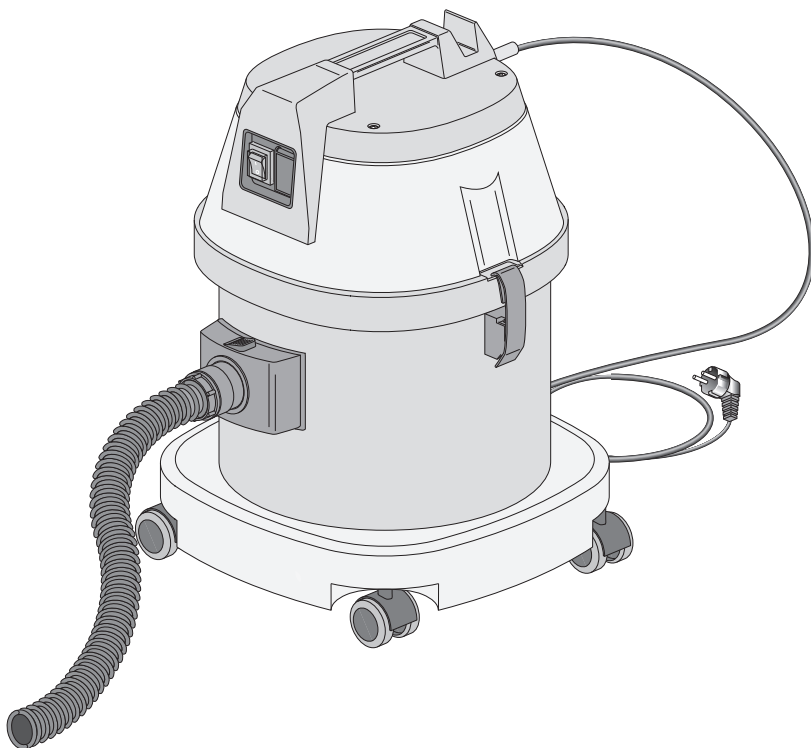

USER MANUAL
Extractor

IT | FR | DE | EN | ES

Zhermack 
Dental

IT **Pag. 5**

FR **Pag. 33**

DE **Pag. 59**

EN **Pag. 85**

ES **Pag. 111**

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PREMESSA

CAP. 1

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INTRODUCTION

CH. 1

Pag. 85	1.1	How to read and use the manual
Pag. 86	1.2	Terminology and symbols

GENERAL INFORMATIONS

CH. 2

Pag. 87	2.1	Responsibility
Pag. 87	2.2	Warranty: general instruction
Pag. 88	2.3	Authorized personnel
Pag. 88	2.4	General safety regulations

DESCRIPTION OF THE EQUIPMENT

CH. 3

Pag. 90	3.1	General description of the equipment
Pag. 90	3.2	Table of the components
Pag. 91	3.2.1	Description of the components
Pag. 91	3.3	Identification data

TECHNICAL CHARACTERISTICS

CH. 4

Pag. 92	4.1	Technical characteristics and operating start
Pag. 92	4.2	Technical Data
Pag. 92	4.2.1	Standard accessories and accessories available on request

TRANSPORT AND MOUNTING

CH. 5

Pag. 93	5.1	Warnings
Pag. 93	5.2	Packing and unpacking
Pag. 94	5.3	Transport and mounting
Pag. 94	5.4	Elimination / Dismantling

INSTALLATION AND FIRST-TIME USE

CH. 6

Pag. 95	6.1	Warnings and precautions
Pag. 95	6.2	Permitted ambient conditions
Pag. 95	6.3	Required work space
Pag. 96	6.3.1	Worktops
Pag. 96	6.4	Connecting up to the power supply
Pag. 97	6.6	Starting up
Pag. 97	6.7	Purpose of the equipment / Improper and forbidden use

USING THE EQUIPMENT**CH. 7**

Pag. 98	7.1	Warnings
Pag. 98	7.2	Safety system
Pag. 98	7.3	Using the aspirator
Pag. 98	7.4	Residual risk

MAINTENANCE**CH. 8**

Pag. 99	8.1	Cleaning of the equipment
Pag. 99	8.1.1	Changing the paper bag filter
Pag. 100	8.2	Routine maintenance
Pag. 100	8.3	Unscheduled maintenance
Pag. 101	8.4	Restarting after a long period of inactivity
Pag. 101	8.5	Requesting technical assistance

TROUBLESHOOTING**CH. 9**

Pag. 102	9.1	Warnings
Pag. 102	9.2	Problems, causes and remedies

APPENDICES

Pag. 104	A.1	EC declaration of conformity
Pag. 105	A.2	Warranty certificate and identification form
Pag. 106	A.3	Maintenance and repair worksheet
Pag. 107	A.4	Authorized service partners
Pag. 107	A.5	Notes

The data given in this manual may be modified without prior notice.

CHAPTER 1: INTRODUCTION

HOW TO READ AND USE THE MANUAL **1.1**

This Use and Maintenance Manual provides information on the utilisation, installation and maintenance of the SAB 1000 aspirator, manufactured by Zhermack.

- The aspirator must be used in accordance with the instructions in this manual: it is therefore recommended that you read it carefully prior to installation and start-up. Make sure that you read the entire manual and pay particular attention to messages written in bold type or items highlighted by boxes and/or symbols: the symbols are designed to highlight situations involving danger or requiring caution (see paragraph 1.2).
- Observance of standards and regulations shown in this manual will ensure that the equipment is used properly and that maintenance work is carried out safely.
- This Use and Maintenance Manual is an integral part of the equipment: it should therefore be kept together with the equipment so that it can be consulted as and when necessary (keep the booklet in a secure dry place away from sunlight, atmospheric agents, etc.). It must be available for consultation throughout the working life of the equipment even when the equipment is sold or finally dismantled.
- It is recommended that you keep this manual constantly updated by integrating any amendments, additions or modifications made by the Manufacturer. Any notes or comments should be made on the blank pages at the rear of the booklet (see Appendices: Notes).
- Use the manual properly so that it is not damaged in any way.
- Do not remove, tear or write on any part of the pages.
- If the manual is lost or damaged and its contents become illegible you can request a replacement copy from the Manufacturer.

The purpose of this Use and Maintenance Manual is to provide users with information on how to operate the equipment properly. The manual contains all the information needed to use the device as intended: in particular the manual includes instructions and information on the following:

- correct installation of the SAB 1000 aspirator;
- detailed description of how the equipment works and its components;
- initial start-up;
- scheduled maintenance;
- basic Safety and Accident Prevention info.

This manual contains nine chapters each of which covers a specific argument. The appendices at the rear of the manual are a useful complement to these chapters.

1.2 TERMINOLOGY AND SYMBOLS

It is important that you understand the following symbols and their significance as they highlight important information such as critical situations, practical advice, or straightforward information.

If you have any doubts as to the significance of a symbol always consult this page.

- Tasks for which failure to observe instructions or tampering with machine parts may put personnel in serious danger are highlighted by the symbol:

**DANGER!**

This symbol refers to safety standards which must be observed at all times in order to protect both workers and machine.

- Critical situations concerning dangerous areas or tasks in which failure to observe the instructions may render the warranty null and void, or cause damage to machine, connected components or the surrounding area, are highlighted by the symbol:

**WARNING!**

This symbol refers to safety standards which must be observed carefully in order to guarantee your safety, other people's safety and prevent damage to the machine.

- Any work that fails to comply with instructions in this manual, or any tampering which might automatically render the warranty null and void, is highlighted by the following symbol:

**FORBIDDEN!**

This symbol refers to actions which must never be carried out (i.e. forbidden actions).

- Any general information and/or advice which may be useful at any time is highlighted by the following symbol:

**NOTE!**

This symbol highlights information and/or useful advice.

CHAPTER 2: GENERAL INFORMATIONS

RESPONSABILITY 2.1

Failure to observe the instructions in this Use and Maintenance Manual exonerates the Manufacturer from any liability whatsoever. For any matters not covered by this manual or about which you have any doubts please contact the Manufacturer directly:

Zhermack S.p.A.
Via Bovazecchino, 100
45021 Badia Polesine • RO • Italy
Tel. +39 0425 597 611 • Fax +39 0425 53 596
<http://www.zhermack.com> e-mail: info@zhermack.com

If equipment maintenance work fails to comply with instructions or is done in such a way as to compromise equipment integrity or modify its characteristics Zhermack cannot in any way be held liable for any liability as regards worker safety and/or equipment performance.

WARRANTY: GENERAL INSTRUCTION 2.2

Included in the Appendices to this Use and Maintenance Manual you will find the Warranty certificate and the ID Form which must be filled out in full.

In general the Warranty is rendered null and void by the following:

- Improper equipment use.
- Incorrect installation.
- Serious negligence of maintenance schedules.
- Modifications to or work on the machine which has not been authorised by the Manufacturer (especially on safety devices).
- Use of non-original spare parts.

IMPORTANT: Always keep the original product packaging throughout the guarantee period. You will need the packaging if the product has to be returned to the Service Centre for repairs or replacement in the event of manufacturing defects or malfunction (with the exception of wilful damage, negligence, failure to perform routine maintenance or damage during transport). If you are not in possession of the original packaging, contact Zhermack and ask for instructions (possible repackaging costs will be charged on delivery). The customer is responsible for all costs associated with unsuitable packaging of the product to be returned to the manufacturer. Zhermack is not liable for faults, malfunctions or damage to the product deriving from or consequent to the use of unsuitable packaging, or packaging not of a quality equal to or better than the packaging used for the original shipment.

Zhermack cannot, as of this moment, be held liable if the machine is modified or tampered with without prior written consent from the Manufacturer.

Consequently, any repairs made by unauthorised personnel, use of non-original spare parts or failure to comply with the installation standards specified in this manual shall immediately render the Warranty null and void.

2.3 AUTHORIZED PERSONNEL

The operator-user must only use the equipment when it is fully assembled and the safety devices are working properly as described in this manual.

Before using this equipment or carrying out any work on it, the operator must read the entire contents of this manual. This manual has been drawn up to provide the user with essential information on rational, safe utilisation of the equipment. The user must comply with the information given at all times. Furthermore, the user must use the work sheets enclosed with this manual so as to keep a record of maintenance/component replacement work as well as make notes on any equipment malfunctions.

2.4 GENERAL SAFETY REGULATIONS

The operator must read the advice and warnings given below and comply with such regulations at all times: doing so will ensure safe, long-lasting equipment performance:

- Do not allow untrained personnel to use this equipment.
- Ensure that the work area is properly prepared: it must be free from obstacles, clean and properly illuminated.
- Take measures to ensure that you are not distracted when using the equipment.
- In the event of a malfunction follow the advice given in this manual: if you have any doubts shut down the equipment and contact your Authorised Service Centre immediately.
- Never use a malfunctioning equipment and always inform the Maintenance Manager of any faults.
- Do not open compartments containing electrical parts.
- Do not replace the power lead or tamper with the supplied plug.
- It is forbidden to use the equipment on items other than those for which it has been specifically designed.
- Before doing any cleaning, routine and/or unscheduled maintenance work make sure that the equipment has been unplugged from the mains socket.
- Make sure that there are no foreign objects inside the work area as these could damage the equipment and/or injure personnel.
- Do not use petrol or flammable solvents as detergents; use only non-flammable, non-corrosive, and non-toxic substances (see paragraph 8.1);
- Do not use compressed air to clean small pieces; if you do so you should protect your eyes with goggles that provide lateral protection and limit air pressure to a maximum of 2 bar.

Zhermack cannot be held liable for any damage to persons or objects caused by incorrect maintenance carried out by unqualified personnel or maintenance work that does not comply with the instructions in this manual.

Zhermack is, as of this moment, exonerated from any liability concerning injury to persons or damage to objects that may result from using this equipment in a way other than that described in this manual.

This equipment has electrical parts: therefore, in the event of a fire, no matter how small, use only powder-type extinguishers. Never attempt to put out an electrical fire with water.


CHAPTER 3: DESCRIPTION OF THE EQUIPMENT

3.1 GENERAL DESCRIPTION OF THE EQUIPMENT

The equipment described in this manual is the SAB 1000 aspirator designed to clean the sandblasting machine.

3.2 TABLE OF THE COMPONENTS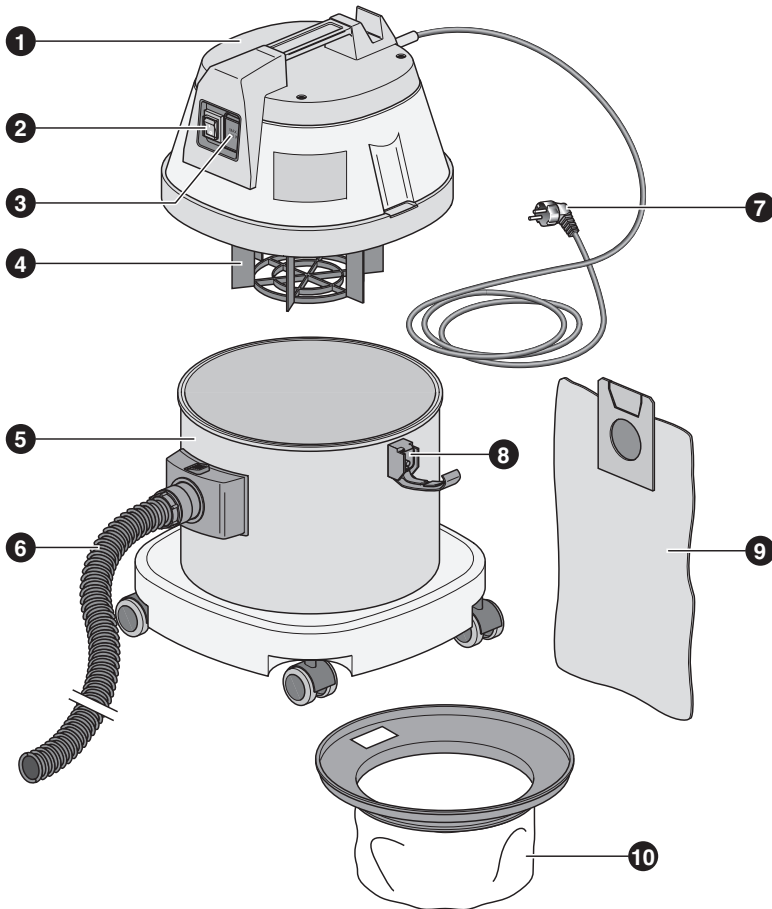


Description of the components 3.2.1

- | | |
|--|--|
| <ol style="list-style-type: none"> 1. Motor unit 2. Main switch 3. Aspirator intake 4. Motor cage 5. Aspirator body | <ol style="list-style-type: none"> 6. Aspirating pipe 7. Mains or the sandblaster connection plug 8. Aspirator closure clips 9. Paper bag filter 10. Felt filter to protect motor |
|--|--|

IDENTIFICATION DATA 3.3

The equipment described in this manual has an ID platelet giving information about both the equipment and the manufacturer:

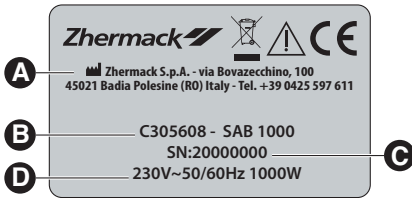- A. Name and address of manufacturer
- B. Equipment code and model
- C. Serial number
- D. Power supply information

The equipment is built in compliance with the EC directives listed in the enclosed "EC declaration of conformity". When requesting spare parts, information or assistance from your Authorised Servicing Centre always quote the data on the ID plate.

Should the ID plate deteriorate and become illegible, even partially, order a replacement plate from the manufacturer: remember to quote the relevant data.

Do not remove and/or damage the plate as you must be able to read the data at all times.

CHAPTER 4: TECHNICAL CHARACTERISTICS

4.1 TECHNICAL CHARACTERISTICS AND OPERATING START

The SAB 1000 aspirator has been designed to clean the sandblasting equipment. The high power SAB 1000 aspirator, fitted with a protective shell to maximise safety, allows filtration as fine as 0.3 mm. It features a by-pass motor and a supplementary switch.

4.2 TECHNICAL DATA

Technical data concerning the **SAB 1000 aspirator**:

• Power supply	230V~ ±10% 50/60Hz
• Rated power	1000 Watt
• Vacuum	190/210 m/bar
• Air flow-rate	150 m ³ /h
• Tank capacity	10 l
• Safety rating	II
• Lead length	10 m
• Aspirating pipe length	2,5 m
• Height	56 cm
• Diameter	Ø 42
• Weight	9 kg
• Noise level	≤ 60 dB

4.2.1 Standard accessories and accessories available on request

In addition to this Use and Maintenance Manual the **SAB 1000** comes complete with:

COMPONENTS	ORDER CODE*
Package of 10 Paper bag filters **	XR0060010
Felt Filter	XR0090030

* Please quote the order code when ordering a spare part. You may order parts from your local Authorised Servicing Centre.

** The aspirator comes complete with one of paper bag filters already fitted.

CHAPTER 5: TRANSPORT AND MOUNTING

WARNINGS 5.1

To prevent any injury to persons or damage to things always proceed with the utmost care and attention when handling the equipment. Observe all the precautions and instructions contained in the following paragraphs.


PACKING AND UNPACKING 5.2

All the material is thoroughly checked by the Manufacturer prior to shipment. When you receive the goods make sure they have not been damaged during transit and that no-one has tampered with the packaging and removed any of the parts contained therein.

Should there be any damage or missing parts you must inform your Dealer where the equipment was purchased immediately. The Carrier must also be informed. Photographic evidence of such damage/missing parts must be provided. The packaging, consisting of a cardboard box with internal expanded plastic padding, contains:

- the aspirator SAB 1000;
- the accessories (see section 4.2.1);
- this Use and Maintenance Manual.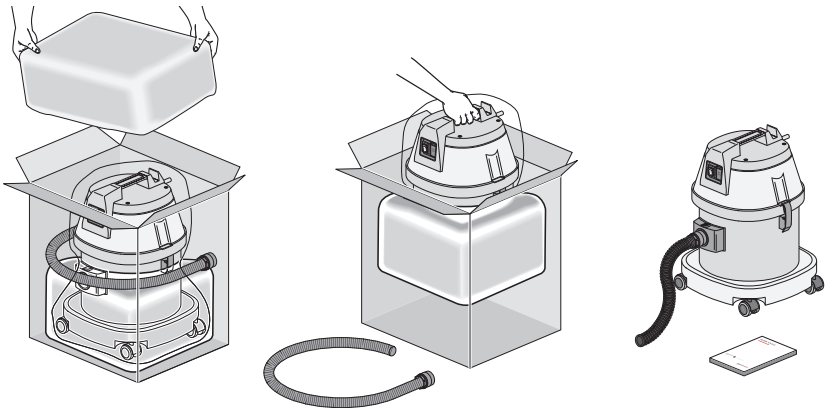

IMPORTANT: Always keep the original product packaging throughout the guarantee period. You will need the packaging if the product has to be returned to the Service Centre for repairs or replacement in the event of manufacturing defects or malfunction (with the exception of wilful damage, negligence, failure to perform routine maintenance or damage during transport). If you are not in possession of the original packaging, contact Zhermack and ask for instructions (possible repackaging costs will be charged on delivery). The customer is responsible for all costs associated with unsuitable packaging of the product to be returned to the manufacturer. Zhermack is not liable for faults, malfunctions or damage to the product deriving from or consequent to the use of unsuitable packaging, or packaging not of a quality equal to or better than the packaging used for the original shipment.

5.3 TRANSPORT AND MOUNTING

Should it be necessary to move the aspirator proceed as follows:

1. Disconnect the equipment from the electrical power supply.
2. Empty the sand and/or work residue from the bag (see Chap. 8) and disconnect the sandblasting machine connection pipe.
3. Hold the bottom of the equipment, making sure that it is always kept vertical.
Proceed cautiously, bearing in mind the size and weight of the unit, so as to prevent any accidents that might be caused by failure to evaluate such factors.
4. Pack the equipment in its original box or, if this is not possible, take all the necessary precautions to protect the equipment against knocks or drops, in that the goods are transported at the owner's risk.

Failure to observe the above exonerates the Manufacturer from any liability concerning equipment malfunction and consequently renders the Warranty null and void.

When shipping/delivering the equipment to your Authorised Servicing Centre always enclose a copy of the purchase document and a copy of the properly compiled ID form.

If the unit is to be transported by courier, the postal service or rail it is recommended such shipments be insured.

5.4 ELIMINATION / DISMANTLING

When disposing of the **packaging materials** the user must comply with the standards in force in his/her country regarding the following materials:

- Wood/paper: non-polluting materials, but must be recycled properly.
- Polystyrene/plastic: pollutants which must not be burnt (toxic fumes) nor dispersed into the environment, but disposed of in compliance with the standards in force in the country of use.

If the equipment is to be **dismantled** the user must, in compliance with EC directives and the laws in force in the country of use, effect elimination and recycling of the following materials:

- Plastic and glass parts, insulated electrical wiring, rubber parts.
- There are no toxic or corrosive parts.

This product must not be disposed of as household waste but rather, when no longer used, must be collected separately according to UE Directive 2012/19.

CHAPTER 6: INSTALLATION AND FIRST-TIME USE

WARNINGS AND PRECAUTIONS **6.1**

Before proceeding with installation make sure that all relevant safety conditions prevail and follow the instructions below carefully.

So as to provide greater clarity the numbers given in the diagrams in this chapter correspond exactly to the numbering in the Components Table (Ch. 3).

PERMITTED AMBIENT CONDITIONS **6.2**

Unless stated otherwise at the time of order the equipment will be configured to operate properly under the following ambient conditions:

Place of Use	Indoors
Altitude	Up to 2000 m
Working temperature	From 5°C to 40°C
Humidity	Max 80%

Ambient conditions other than those listed above may cause malfunctions or sudden breakdowns. Lighting in the installation area must be sufficient to provide good visibility at every single point on the equipment.

More specifically, luminosity must not be less than 200 lux, lighting must be as uniform as possible and there must be no reflected light as this could dazzle the operator.

The equipment must not be used in workplaces having an atmosphere which is explosive and/or at risk of fire as it has not been designed for use in such areas. Nevertheless, should a fire accidentally break out follow the procedure described in paragraph 2.4.

Work area lighting is an important factor in both personnel safety and the ultimate quality of the work being done. In Italy lighting must comply with a Ministerial Decree Law that clearly defines minimum lighting requirements. In other countries lighting requirements form part of the accident prevention and workplace hygiene standards.


REQUIRED WORK SPACE **6.3**

Choosing a good workplace with an appropriate amount of space available for equipment installation is an essential factor in personnel safety, the quality of the work and proper maintenance. This zone must not only be spacious enough to allow for optimum equipment operation but must also be well illuminated, aired, not dusty and not exposed to direct sunlight. Note also that the unit must be positioned so the connection plug can be handled/manipulated with ease.

6.3.1 Worktops

The SAB 1000 aspirator has been designed to be placed directly on the floor.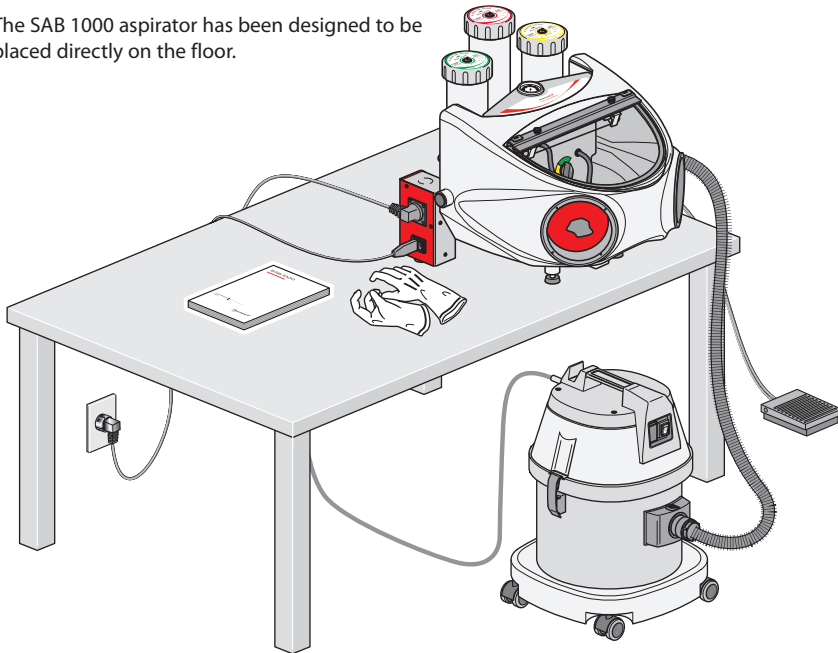

6.4 CONNECTING UP TO THE POWER SUPPLY

The user must provide a power connection with relative socket near the equipment installation point. The user must also install an adequate electrical circuit breaker upstream from the socket as well as efficacious overload/indirect contact safety devices.

Connection is effected via the safety plug (16A, in compliance with European standards) at the end of the power lead: this must be inserted in the mains socket.

When connecting check:

- that mains voltage and frequency are as indicated on the identification plate (incorrect power supply voltage may damage the equipment).
- that the mains supply has a proper, efficient earth connection.

It is forbidden to tamper with the power lead and its plug. When replacing them owing to damage and/or wear please contact an Authorised Service Centre.

A proper earth connection is compulsory. Should it temporarily be necessary to use an extension lead the latter must comply with the standards in force in the country of use.

STARTING UP 6.5

Fig. 1: After positioning the aspirator on the floor correctly (see section 6.3.1) insert the plug (7) in the embedded socket (X) on the right-hand side of the sandblaster.

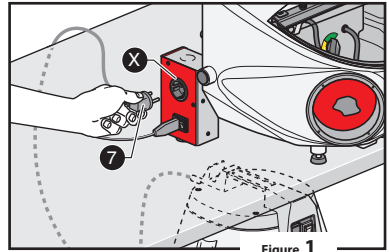
Figure 1

Fig. 2: Connect up the aspiration tube (6) to the connector on the right-hand side of the sandblaster (Y).

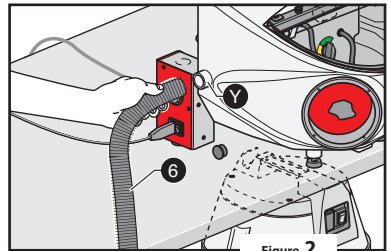
Figure 2

Failure to observe any of the above equipment installation instructions may lead to malfunctions and render the warranty null and void.

PURPOSE OF THE EQUIPMENT / IMPROPER AND FORBIDDEN USE 6.6

The SAB 1000 aspirator has been designed to perfectly clean the sandblasting machine.

Using the equipment for any purpose other than that described in this Manual shall be considered improper and thus forbidden.

Improper use renders the Warranty null and void and the Zhermack shall not be liable for any damages to objects, workers or third parties.

CHAPTER 7: USING THE EQUIPMENT

7.1 WARNINGS

The equipment must not be used with any of its parts disassembled: before using the equipment make sure that all its components are properly in place.

To ensure that work is done in absolute safety always make sure that you follow the described procedures with caution.

To provide greater clarity the numbers given in the diagrams in the following paragraphs correspond exactly to the numbering in Chapter 3 in the "Equipment Description" section.

7.2 SAFETY SYSTEMS

In compliance with EC 89/392 international accident prevention standards, these equipments are fitted with safety devices such as fixed and mobile guards to shield moving parts. This ensures operator safety when the equipments are in use.

7.3 USING THE SAB 1000

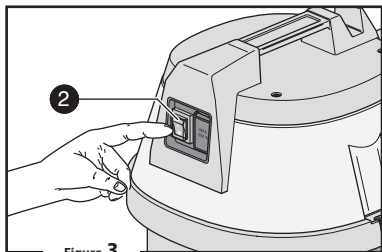
Figure 3

Fig. 3: After positioning the aspirator on the floor correctly (see Chap.6), switch on the equipment by pressing the main switch (2).

Aspirator operation is interlocked with sandblaster operation: the aspirator starts working when the sandblaster is switched on.

7.4 RESIDUAL RISK

Residual risk is:

- a danger that cannot be completely eliminated through design and application of safety devices;
- a potential danger which cannot be highlighted;
- work carried out on the equipment interior by unauthorised personnel.

To prevent accidents always observe the instructions in this manual. Should you have any doubts at all please contact the manufacturer Zhermack or your Authorised Servicing Centre.

CHAPTER 8: MAINTENANCE

CLEANING OF THE EQUIPMENT 8.1

The equipment exterior can normally be kept clean just by wiping it with a dry cloth. If necessary, wet the cloth just a little water or use a non-degreasing detergent.

The operator must ensure that the equipment is kept clean and free from any foreign bodies such as dirt, water, non-conducting liquids etc.

The equipment must be cleaned at the end of every work session to remove all residues. Cleaning must be done with the equipment at standstill and positioned in a stable manner.

Before cleaning the equipment exterior always turn the main switch to OFF. It is strictly forbidden to clean the equipment while it is connected to the mains power supply.

Never use inflammable, corrosive, alkaline or toxic liquids when cleaning the equipment.

Eye goggles and a face mask must be worn when using compressed air for cleaning purposes. Do not allow any persons near the equipment as they could be hit by flying dust.

Changing the paper bag filter 8.1.1

To change the paper bag filter on the SAB 1000 aspirator (usually changed weekly) proceed as follows:

- switch off the aspirator at the main switch and disconnect it from the sandblasting machine;
- open the aspirator by acting on the clips (8) and remove the motor unit (1);
- remove the felt filter (10) and detach the aspiration pipe (6) from the coupling;
- remove the bag (9) by detaching the inlet from the bag coupling.

When carrying out this task proceed carefully so as to prevent dust entering the interior of the aspirator: if dust does get in, clean thoroughly.

- Insert the new bag on the relative coupling via the inlet;
- reposition the felt motor protection filter (10) so that its edges rest on the aspirator body (5);
- place the motor unit (1) above the aspirator body (5), checking that the felt filter is in its seat on the motor unit;
- close the aspirator by acting on the clips (8), previously positioned in their housings;
- re-insert the aspiration pipe (6) in the coupling;
- reconnect the plug (7) to the sandblasting unit socket, then switch on via the main switch (2).

8.2 ROUTINE MAINTENANCE

Once the equipment has been cleaned (see section 8.1) the operator must check for worn, damaged or loose parts. If any such anomalies are discovered contact your Authorised Servicing Centre.

Should any of the above anomalies be discovered it is strictly forbidden to restart the equipment before such faults have been set right.

More precisely, should the operator note any defects or problems of any kind whatsoever he/she must place a warning sign on the equipment indicating that maintenance work is in progress and that using the equipment is forbidden (EC-compliant signs can be purchased from the relevant suppliers). Routine maintenance, cleaning and proper equipment use are essential factors in guaranteeing long-lasting equipment performance and safety.

Equipment safety devices/guards must never be removed except for repair and maintenance purposes. Such items must be replaced as soon as the situation which led to their removal no longer exists and in any case before the equipment is used again.

8.3 UNSCHEDULED MAINTENANCE

Unscheduled maintenance includes repair of any accidental breakage and replacement of worn or malfunctioning parts.

All electrical and mechanical work must only be carried out by an authorised Zhermack maintenance technician or your Authorised Servicing Centre.

Modifications which alter the equipment characteristics from a safety and risk prevention viewpoint may only be made by the Manufacturer who shall then issue a certificate stating that the equipment conforms to safety standards. Therefore any modification or maintenance tasks not actually described in this manual are to be considered forbidden. If the equipment is modified, tampered with or repaired by unauthorised personnel or non-original spare parts are used the manufacturer is relieved of all responsibility. If a maintenance task is required but is not described in this manual you must contact an Authorised Servicing Centre.

RESTARTING AFTER A LONG PERIOD OF INACTIVITY **8.4**

Should the equipment need to be restarted after a particularly long period of inactivity proceed as described in Chapter 6 (First-time Use).

If there are any faults or malfunctions proceed as described in Chapter 9 (Troubleshooting). If the problem persists or is not covered by the information in this chapter contact your Authorised Servicing Centre immediately.

REQUESTING TECHNICAL ASSISTANCE **8.5**

To obtain technical assistance choose one of the following options:

1. contact your Authorised Servicing Centre.
2. telephone the Zhermack Customer Assistance Service on the number **+39 0425.597.611** and proceed as follows: give personal data (e.g. address and telephone n.), state the information on the equipment ID plate (see paragraph 3.3) and explain the problem clearly.
3. fax all the above information and a properly filled out ID Form (see Appendix A.2) to **+39 0425.53.596**.

CHAPTER 9: TROUBLESHOOTING

9.1 WARNINGS

Should any of the anomalies described below occur try setting the situation right, as far as is permissible, by following the instructions contained in this handbook. Should the problem persist contact your Authorised Servicing Centre.

9.2 PROBLEMS, CAUSES AND REMEDIES

PROBLEM	CAUSE	REMEDY
With the main switch set to ON the aspirator does not work.	<ol style="list-style-type: none"> 1. The aspirator plug is not correctly inserted in the sandblaster socket. 2. The sandblaster power lead is not properly inserted in the socket or is faulty (bare, damaged wiring etc.). 	<ol style="list-style-type: none"> 1. Check that the plug is inserted properly. 2. Replace the power lead. Contact an Authorised Assistance Centre only.
The aspirator motor jams.	<ol style="list-style-type: none"> 1. The paper bag filter is full. 	<ol style="list-style-type: none"> 1. Change the paper bag filter. If the problems persists contact an Authorised Service Centre.

APPENDICES

CE DECLARATION OF CONFORMITY **A.1**

WARRANTY CERTIFICATE AND IDENTIFICATION FORM **A.2**

MAINTENANCE AND REPAIR WORKSHEET **A.3**

AUTHORIZED SERVICE PARTNERS **A.4**

NOTES **A.5**

A.1 EC DECLARATION OF CONFORMITY

The Manufacturer:

Zhermack S.p.A.
Via Bovazecchino, 100
45021 Badia Polesine • RO • Italy
Tel. +39 0425 597 611 • Fax +39 0425 53 596

Person authorised to compile the technical file:

Zhermack S.p.A.
Via Bovazecchino, 100
45021 Badia Polesine • RO • Italy
Tel. +39 0425 597 611 • Fax +39 0425 53 596

declares under its sole responsibility that the product:

SAB 1000

is in conformity with Directive 2011/65 EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.


It is also in conformity with the laws implementing the following directives:

- EC Directive 2004/108 EC (EMC Directive) and subsequent amendments.
- EC Directive 2006/95 (Low Voltage Directive) and subsequent amendments.

Please note that the manufacturer's liability for damages caused by a defective product expires 10 years from the date on which the manufacturer released such product on the market and that such liability is transferred to the user in accordance with Directive 85/374/EEC, subsequent amendments and the laws implementing it.

Badia Polesine, 25/02/2014

Paolo Ambrosini
General Manager

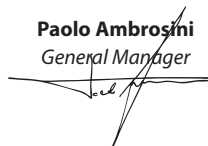WARRANTY CERTIFICATE AND IDENTIFICATION FORM **A.2**

1. This document certifies that the Manufacturer has built the product correctly using only top quality materials, carried out all the necessary tests and that the product complies with EC directives. This product Warranty is valid for 12-months as from date of delivery to the customer. Proof of such date shall be provided by a purchase document or the completed identification form. These documents must be handed over together with the equipment to the Servicing Centre. Persons wishing to make use of warranty coverage must make the fault known within eight days of its discovery as per Italian civil code art. 1495. Warranty cover is limited to replacement or repair of individual components or pieces with manufacturing defects and does not cover costs related to transfer of technical personnel, transport, packing etc. Breakdowns, faults or damages which derive from poor maintenance, incorrect power supply, negligence, inexperience and other causes not imputable to the Manufacturer and parts subject to normal wear are not covered by the Warranty. The Warranty does not cover breakdowns caused by failure to carry out routine maintenance or customer negligence. This Warranty does not provide for any awards for direct or indirect damages of any kind caused to persons or things by any equipment inefficiency.
2. The Warranty is instantly rendered null and void if the equipment is repaired, modified or tampered with by the purchaser or any unauthorised third parties. Responsibility for damages caused by defective products expires 10 years after the Manufacturer puts them on the market: after such time damages are the user's responsibility, as stated in EC Directive 85/374.
3. To have work carried out under Warranty the purchaser must only contact the retailer, the Manufacturer-authorized servicing centres, or the Manufacturer. The Warranty entitles the purchaser to free replacement of defective parts. It does not entitle the purchaser to replacement of the entire equipment.
4. In the event of disputes over application of the Warranty, the quality or condition of the equipment, the purchaser shall not suspend payment or delay payment of the price or price instalment.
5. The purchaser shall not be entitled to damages for any loss of production.
6. The Warranty is rendered null and void if:
 - a. the equipment is damaged by a fall, exposed to fire, spilt liquids, lightning, natural disaster or any other cause not imputable to the Manufacturer.
 - b. installation has not been carried out properly.
 - c. power supply is incorrect (e.g. rated mains voltage incorrect). Erroneous mains connection (rated mains voltage incorrect), or failure to install proper protection devices.
 - d. the serial number has been removed, cancelled out or altered.
7. Any components to be replaced under the terms of the Warranty shall be sent back to Zhermack S.p.A. which shall then send the appropriate spare parts. Should the changed part not be replaced it shall be charged to the person who ordered it.
8. The Manufacturer and the Dental Deposit are not required to provide replacement equipments while repairs are carried out.
9. Spare parts shall, for tax purposes, only be provided under warranty where the relevant limitations set out in the terms and conditions of the warranty are observed.
10. For any other eventuality not provided for by this Warranty Certificate and the regulations consult the Italian Civil Code.
11. Labour, travel and call-out invoices must be paid for on their receipt.
12. The manufacturer and the dealer shall comply with all data processing laws, including proper observance of data security standards in compliance with the specifications given in the data processing information sheet.

IDENTIFICATION FORM	
EQUIPMENT MODEL:	EQUIPMENT SERIAL NUMBER:
PURCHASER:	ADDRESS:
TOWN / POSTCODE:	TEL:
TAX CODE / VAT NUMBER RETAILER:	RETAILER:
	PURCHASER'S SIGNATURE:

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